



The UVC 254 TRI Card provides a visual indication of germicidal irradiation at three dose levels: 25, 50 and 100 mJ/cm². Using a patented photochromic ink, the yellow center on the card reacts to exposure from UV-C devices at 254 nm and changes color to indicate the intensity of the dose delivered.

The single-use cards feature an adhesive strip for easy placement on a variety of surfaces and have writeable areas for date, time and location. For best results, take exposure readings immediately after a disinfection cycle and notate the card by circling the dose level achieved.

UVC Dosimeters are an important tool to achieve optimal UV-C disinfection by enabling operators to:

- Determine the dose delivered at varying distances from the device
- Identify shadowed areas that may require device repositioning
- Optimize run-times for efficiencies in disinfection cycles
- Provide evidence of disinfection cycles for credentialing, auditing and data collection



INTELLEGO
TECHNOLOGIES

UVC Dosimeters are
manufactured by Intellego
Technologies AB, Sweden



VTM (UK) LTD

UVC 254 TRI Card

Designed for use with UV-C 254 devices

Understanding Dose & Pathogen Inactivation

To achieve successful UV-C disinfection, it is critical to identify how much germicidal irradiation is delivered to a surface. There are no known pathogens that are resistant to UV-C, but inactivation is dependent on the amount of UVGI delivered.

Studies have shown that a dose of 22 mJ/cm² can achieve a 99.999% reduction of SARS-CoV-2, and doses of 50 and 100 mJ/cm² have been correlated to a 99.9% reduction of MRSA and C. Difficile, respectively. UVC Dosimeters provide simple, visible evidence so you can see if your disinfection cycle is successful.

A compilation of studies showing UV doses required for inactivation of bacteria, viruses, spores and fungi is available at iuva.org/Guidance-Documents.




- For use with 254 nm devices (low-pressure mercury vapor lamps)
- Provides visible evidence of UVGI at 25, 50 and 100 mJ/cm²
- Recommended for use with every disinfection cycle
- Ideal for staff training, validating performance, and comparison of different UV-C devices
- Low-cost, easy-to-use
- Clinically proven accuracy and reliability
- 3-year shelf life
- Designed and manufactured in Sweden

According to a 2021 study published in Infection Control & Hospital Epidemiology, the researchers concluded:

“There is a need for practical tools for monitoring doses delivered by UV-C devices. Our results suggest that colorimetric indicators could be useful tools to compare different devices, assess delivery of UV-C to different sites in patient rooms and confirm that in-use devices are operating correctly.”

Order Information

To learn more or request pricing, visit UVCdosimeters.com or email sales@uvc-vtm.co.uk

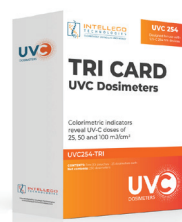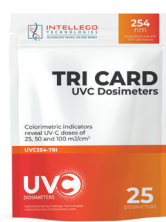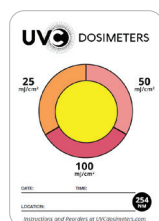

UVC 254 TRI Card

Product SKU

UVC254-TRI

Packaging

25 Dosimeters per Pouch

10 Pouches per Box

VTM UK LTD
Unit 6 Nash Hall
The Street
High Ongar
Essex CM5 9NL

Tel: 01462 473820
Email: sales@uvc-vtm.co.uk
Web: www.uvc-vtm.co.uk



INTELLEGO
TECHNOLOGIES

UVC Dosimeters are designed to provide a visual indication of applied UV-C energy during a disinfection cycle and are not a substitute for microbiological verification of disinfection. Store at room temperature and away from direct sunlight. **Do not reuse.**